



MICRO BLOOD COLLECTION SYSTEM



**SAFE
CAP[®]**

Self-sealing microhematocrit capillary tubes with

SafeWrap

Unique, self-sealing capillary tube with Mylar wrap minimizes risk of exposure to HIV, Hepatitis, and other bloodborne pathogens.




If accidental breakage occurs, **triple layered Mylar overwrap** minimizes exposure to bloodborne contaminants and glass fragments.

SAFECAP[®]



1. Fill: Factory installed plug has channel allowing air to escape while filling the tube.

2. Hold Vertical: Channel seals automatically when blood touches plug.

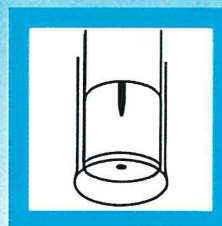

3. Centrifuge and Read

This simple procedure produces additional safety benefits by eliminating normal fingertip contact, the hazards of manual sealing, and the use of contaminated sealants. It's that simple... it's that safe!



Product Ordering Information

SAFECAP[®] Self-Sealing Mylar-Wrapped Capillary Tubes

CATALOG #	DESCRIPTION	QUANTITY
SAFECAP[®] Self-Sealing Mylar-Wrapped Capillary Tubes		
H75-2000M	Heparinized Self-Sealing with Mylar Wrap	Case of 20 Vials contains 100 tubes each. 75 mm length.
P75-2000M	Plain Self-Sealing with Mylar Wrap	Case of 20 Vials contains 100 tubes each. 75 mm length.
R60-2000M	Precalibrated Self Sealing with Mylar Wrap	Case of 20 Vials contains 100 tubes each. 75 mm length. 60 mm length calibrated from tip. Heparinized for use on direct reading centrifuges.

SAFEPETTE[®] Harvesting Dispenser System

SD-2050	Harvesting Dispenser System	Individually packaged
---------	-----------------------------	-----------------------

Tube Organizer

CTO-10	Capillary Tube Organizer	6 each per case
--------	--------------------------	-----------------



Where Innovation Scales

500 Parkway Drive ■ Broomall, PA 19008 ■ www.drummondsci.com

Ordering Information: 1-800-523-7480 ■ orders@drummondsci.com

©2023 Drummond Scientific Company. All Rights Reserved. Printed in USA. US Patent No: 5,065,768. Other Patents Pending
Mylar is a product of DuPont Nemours & Company, Inc.