



## DRUMMOND *Plasticrit*

### 75mm Plastic Capillary Tubes (Heparinized & Plain)

#### Instructions for Use

1. Hold the Drummond *Plasticrit* capillary tube parallel to the ground, and collect your sample using the distal end of the tube.
2. Ensure the sample fills evenly throughout the tube.
3. Insert the opposite end into the clay sealant.
4. Your sample is now ready for testing. Please refer to your platform's test instructions for use and results.

#### Storage Instructions

- The Drummond *Plasticrit* should be stored in a room temperature environment.
- Shelf life of DRUMMOND *Plasticrit* is indicated on the packaging.
- Heparinized / Expiration 24 months
- Plain / Expiration 48 months